

## BRA & PROSTHESIS CARE

### **BRAS:**

Hand wash in warm water with mild soap. Hang until completely dry for best results.

### **PROSTHESES:**

To clean a breast form, wash gently in lukewarm water with a mild soap. Dry with a soft towel - do not wring or rub. Avoid using powder, perfume, or hot water directly on the form.

Care should be taken with sharp objects. Fingernails, pins, broaches, scissors, or your pet's claws/nails can all cause permanent damage to the soft film surrounding the silicone.

It is important to wear the correct bra size. Bras that are too small or with underwires that press into the breast form may crease or permanently damage the form.

It is important to handle the form gently when inserting into the pocket of the bra, as repeated squeezing or pulling can cause damage over time.

When not wearing the breast form it is recommended that it be stored in the cradle of the box provided and away from heat. This will help maintain the shape .