INSTRUCTION MANUAL

4 Wheel Rollator with Locking Brakes

PLEASE READ THIS MANUAL CAREFULLY TO ENSURE SAFE USAGE AND MAINTENANCE:

Contents:

- Rollator frame
- Backrest
- 2 handles with brake cables attached to legs
- Front legs
- 6 triangle bolts

Safety Precautions

- Make sure the rollator is fully opened and the seat is down. Ensure folding bar under the seat is locked in the down position.
- Be sure both handles are at equal heights and secured with triangle bolt.
- Test rollator for stability before applying weight to the hand grip.
- Check fit and tightness of knobs, nuts and bolts on a weekly basis to ensure rollator is safe to use.
- Test loop brakes before operating the rollator. Lock bar on the rear wheels should prevent wheels from turning. If the wheels turn while brakes are locked, please take the unit back to the dealer for adjustment. When locks are released the wheels should turn freely.
- Always lock loop brakes before sitting down.
- Do not use seat when the unit is on an incline or uneven ground.
- Replace tires when damaged or extensively worn.
- This is not a transport device. Do not allow anyone to push you while you are seated on the rollator.
- **Product weight capacity: 300 LBS**

Assembly Instructions

1. Open frame and insert leg 1 in to the frame end 1 (see figure 1). Insert the push pins to the desired height and secure the legs with the triangle bolt.
2. Repeat for connecting legs 2, 3 and 4 on the frame as indicated in step 1. ****NOTE:** Do not use the last hole for the push pin. It is intended for the triangle bolt to secure the legs (see figure 2).
3. Turn rollator over so that the seat faces up. Make sure the leg hinges lock into place.
4. Insert handles into the frame at desired height lining up holes appropriately. Slide bolts through holes from the inside and screw triangle bolt until secure.
5. Slide padded backrest into frame brackets. Making sure the snaps are secured to the frame.

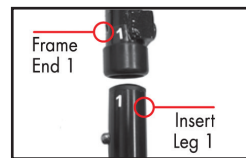
figure 1

figure 2

Loop Lock Brakes

1. To slow down or to stop temporarily, pull up slowly on the loop brakes with four fingers until the unit slows down or comes to a stop.
2. To temporarily lock wheels in position, press down on both loop brakes with palms until both brakes stay on the down position. That will signify that the wheels are locked.
3. To release wheel locks, pull up on both loop brakes with your palm or four fingers to the original position.
4. Always engage and disengage wheel locks on both sides at the same time.

5. To adjust brakes, use a wrench and turn adjustment bolt clockwise.

Height Adjustment

1. Remove tightening knob from handle.
2. Set handle to desired height.
3. Replace tightening knob into handle, making sure to insert knob from the outside.
4. Repeat for second handle, making sure both handles are set to the same height setting.

Leg Height Adjustment

1. Remove tightening knob from leg.
2. Depress push pin and slide leg to desired height.
3. Replace tightening knob into leg, making sure to insert knob from the outside.
4. Repeat for second, third and fourth legs, making sure all are set to the same height setting.

Limited Lifetime Warranty

Roscoe Medical is focused on providing quality products. This product is designed and built to precise standards and is carefully inspected prior to shipment. This product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This warranty does not cover product failure due to owner misuse or negligence or normal wear and tear. This warranty does not extend to non-durable components such as rubber accessories which are subject to normal wear and tear and does require periodic replacement. In the event of a defect, Roscoe Medical will, at our option, replace or repair the product.

If you have any questions, please contact an authorized Roscoe Medical Provider or call Roscoe Medical at the number below.

Roscoe Medical, Inc
Middleburg Heights, OH 44130
www.roscoemedical.com
Questions? Please call: 800-376-7263
Made in China | Hecho en China

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