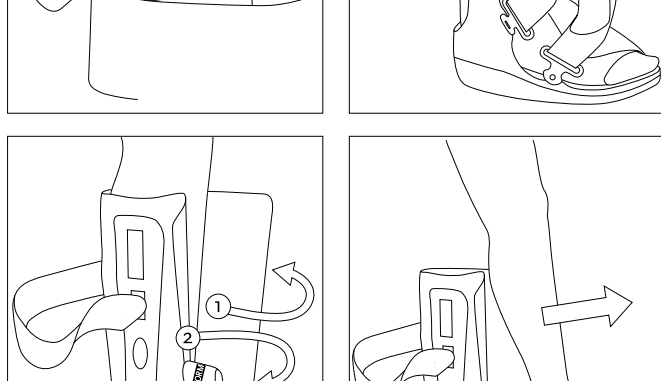
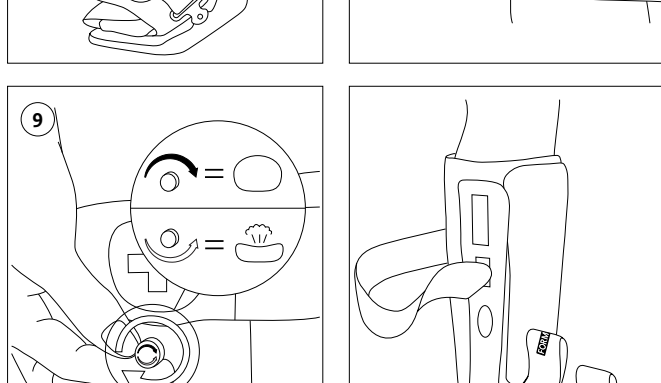
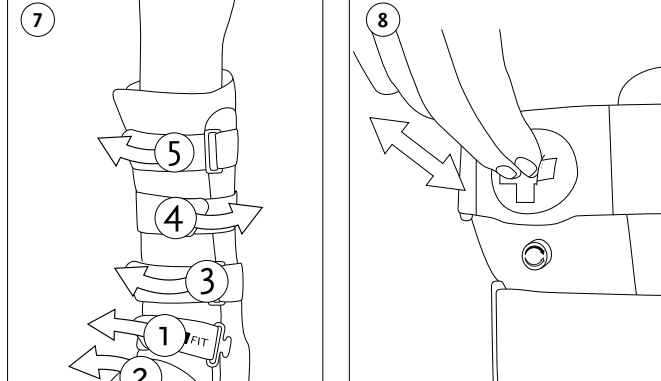
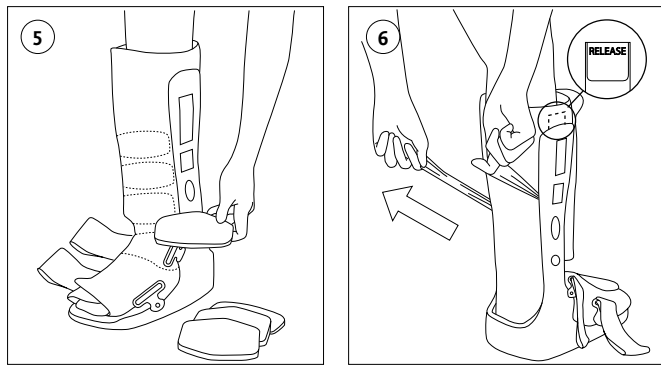
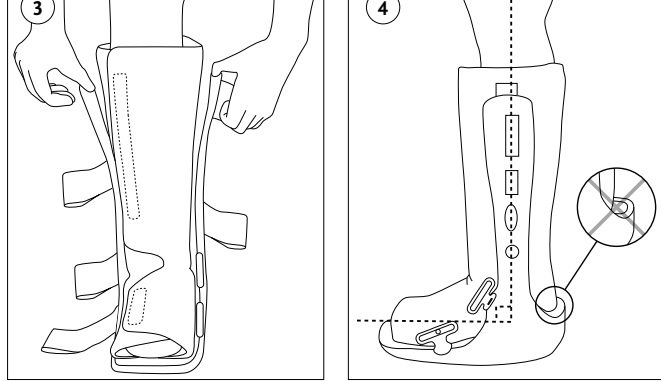
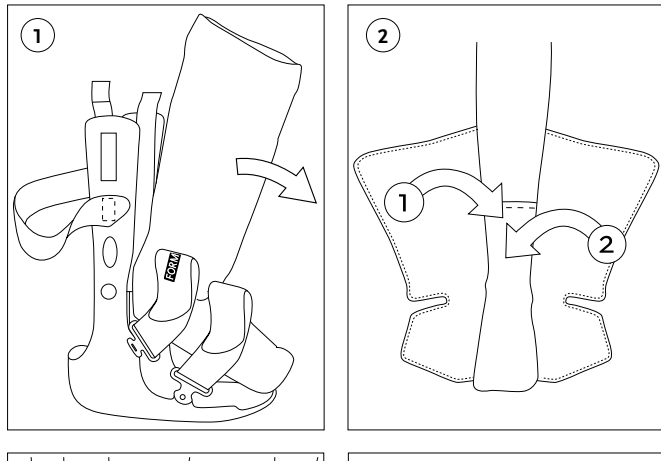
ENGLISH

DESCRIPTION OF SYMBOLS

MD Medical Device

INTENDED USE

The device is intended for immobilization of the foot and ankle.

Indications For Use

High top: Foot and ankle conditions that may benefit from motion restriction, such as: Grade 2 and 3 ankle sprains; Stable fractures; Soft tissue injuries; Post-operative, trauma or rehab use. Low top: Foot and ankle conditions that may benefit from motion restriction, such as: Stable fractures; Soft tissue injuries; Post-operative, trauma or rehab use.

Contraindications

None known.

Warning And Cautions

Caution: Do not remove walker unless instructed by physician.

Caution: Be sure the patient performs circulation checks. If a loss of circulation is felt, or if the walker feels too tight, release control, close straps and adjust to a comfortable level. If discomfort continues, the patient should contact their medical professional immediately.

Caution: When prescribed to patients with diabetes, vascular deficiencies and neuropathies extra caution and attention to patient is needed.

Note: Do not remove the inner foam pads which are adhered to the plastic walker boot.

GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference.

Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

Warning: The patient should immediately contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions;
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

FITTING INSTRUCTIONS

- Unfasten straps and remove liner from walker (Figure 1). Do not remove pulp tabs yet.

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