



**CORFLEX®**

Designed for life.

CORFLEX.COM | 800 426 7353

# Marathon™ Air Walker Tall

## Application Instructions



The Marathon Air Walker features an inflatable interior bladder that allows for a custom fit while providing compression and stabilization throughout recovery.

## Fitting and Application

**1.** Place leg inside liner and position heel as far back as possible. Close forefoot cover over top of foot and fasten it snugly.



POSITION FOREFOOT  
COVER OVER TOP OF FOOT

**2.** Wrap liner around the front of lower leg and secure with hook and loop closures.



WRAP LINER AROUND  
FRONT/LOWER LEG

FOREFOOT COVER

## Fitting and Application (continued)

**3.** Situate front shell over liner taking care to properly position it over the inflation bulb.



Check that the inflate/deflate knob is in the DEFLATE position before tightening straps, this will allow air to escape from the bladder.

The front shell should not contact the foot or leg, adjust as needed.

**4.** Insert ankle strap through plastic buckle and secure strap. Always secure this strap first to encourage proper positioning in the walker.



SECURE ANKLE STRAP  
FIRST FOR PROPER FIT

**5.** Secure forefoot strap. Insert secondary straps through buckles and fasten back on itself.



INSERT SECONDARY STRAPS  
THROUGH BUCKLES

**6.** Test for desired comfort level, do not overtighten straps.



CAUTION:  
DO NOT  
OVERTIGHTEN  
STRAPS

TIGHTEN STRAPS  
UNTIL SECURE AND  
COMFORTABLE

**7.** To inflate liner, turn the inflate/deflate knob clockwise until fully opened and press and release inflation bulb until liner is providing support and comfort.




INFLATE LINER  
CAUTION:  
DO NOT  
OVERINFLATE  
LINER

PRESS AND RELEASE  
INFLATION BULB AND  
TURN THE INFLATE/  
DEFLATE KNOB  
COUNTERCLOCKWISE

# Fitting and Application (continued)

7. To release air from liner, turn the inflate/deflate knob counterclockwise several times. If at any time the liner feels too tight, immediately release air by turning the inflate/deflate knob counterclockwise.

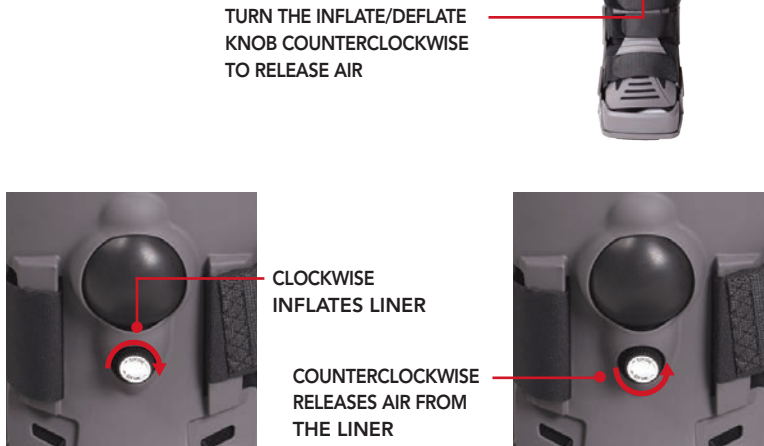

- DEFLATE LINER BEFORE REMOVING THE AIR WALKER.
- PRESSURE CAN BE ADJUSTED BY TIGHTENING AND LOOSENING STRAPS.
- DO NOT OVERINFLATE.
- FOLLOW PHYSICIANS INSTRUCTIONS.

## Marathon Air Walkers

Designed to increase stability and decrease edema, our walker features a built-in pump to achieve desired level of compression. Ideal for stable foot/ankle fractures, severe sprains, soft tissue injuries, post cast removal and post surgical application. Custom sole and low profile non-skid bottom and adjustable toe protector promotes natural gait. Available in two heights, ankle and tall.



## Troubleshooting:

If you cannot inflate or deflate the liner check that the metal valve located on the top of the inflation bulb is not blocked.

If you cannot inflate or deflate the liner check that the inflate/deflate valve is in the correct position.

## Care Instructions:

Liner should be hand washed only using warm water and detergent for delicate fabrics, repeatedly rinsed and line dried only. Do not use bleach or place in a dryer. Wipe plastic shell with damp cloth, you may use disinfectant or mild soap and wipe dry.

## Caution:

This product has been designed and tested based on single patient use only. It should never be used by multiple patients. If additional pain, swelling or symptoms occur while using the device, seek medical attention. Product should be applied by a licensed or certified healthcare practitioner and as prescribed by a physician or other qualified medical authority.

## Indications:

Indicated for stable foot and ankle fractures, severe sprains, soft tissue injuries, post-op and post-cast removal.

## Warranty:

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex.

## To Reorder:

Product Number	XS	S	M	L	XL
Marathon Air Walker Tall	79-2010	79-2031	79-2012	79-2013	79-2014
Replacement Liner Tall	79-2030	79-2031	79-2032	79-2033	79-2034

Sizing Guide	XS	S	M	L	XL
Men's Size	3-4	4½-7	7½-10	10½-12½	13+
Women's Size	4-5½	6-8	8½-11	11½-13	14+

Measured by Shoe Size



**CORFLEX®**  
Designed for life.

CORFLEX | 669 EAST INDUSTRIAL PARK DRIVE | MANCHESTER, NH 03109  
CORFLEX.COM | TOLL FREE: 800 426 7353 | FAX: 603 623 4111