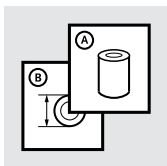
Instructions for Use

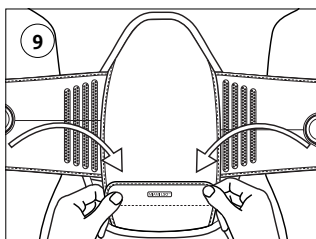
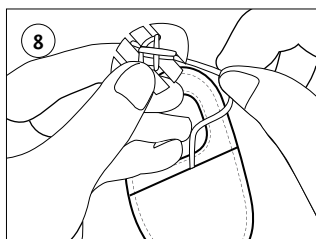
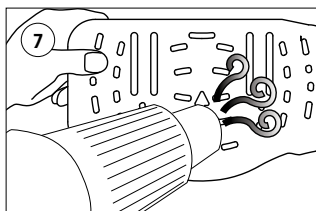
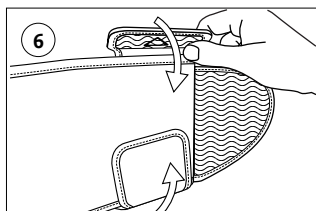
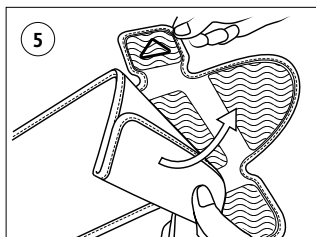
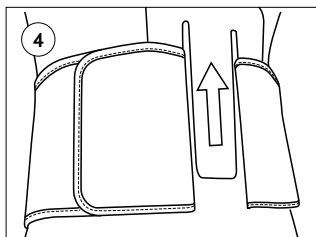
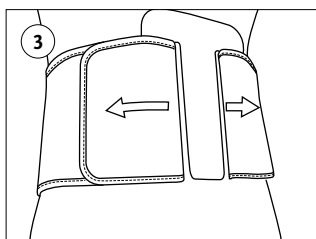
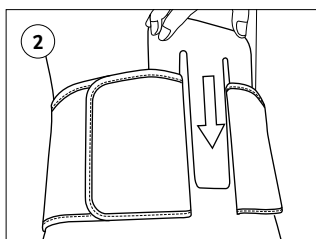
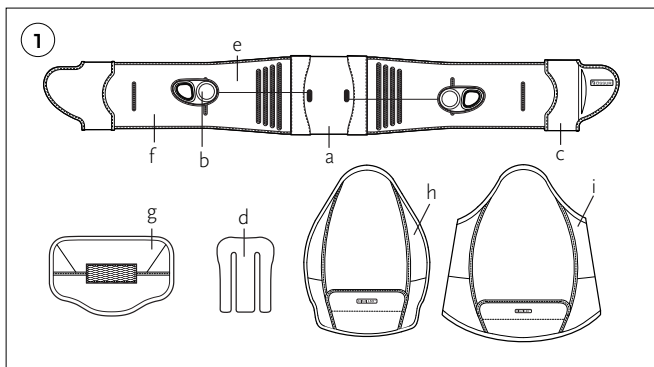
---

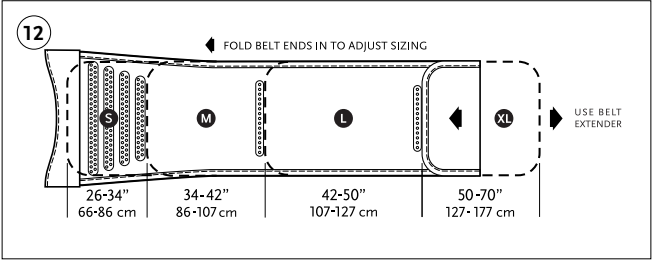
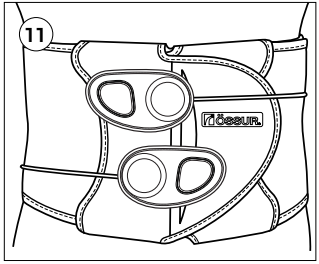
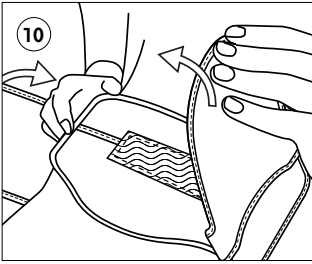
# LUMBOSACRAL FIXATION BRACE

Miami LSO

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	8
FR	Notice d'utilisation	12
ES	Instrucciones para el uso	16
IT	Istruzioni per l'uso	20
NO	Bruksanvisning	25
DA	Brugsanvisning	28
SV	Bruksanvisning	31
EL	Οδηγίες Χρήσης	35
FI	Käyttöohjeet	39
NL	Gebruiksaanwijzing	43
PT	Instruções de Utilização	47
PL	Instrukcja użytkowania	51
CS	Návod k použití	55
TR	Kullanım Talimatları	58
JA	取扱説明書	61
ZH	中文说明书	64
KO	사용 설명서	69

## LIST OF PARTS: (Figure 1)

- a. Belt Compression System
- b. Compression System Handles
- c. Overlap Closures
- d. Össur Fit Tool
- e. Adjustable Belt
- f. Hook and loop Stay Fit Tabs
- g. Anterior Panel
- h. Posterior Panel without Lateral Support
- i. Posterior Panel with Lateral Support

## INDICATIONS FOR USE

Conditions requiring gross immobilization of the trunk in the lumbar region. Such conditions may include stable, non-displaced spinal fractures (L1-L5); Spinal stenosis; Herniated discs; Degenerative spinal pathologies; Spondylolisthesis; Spondylolysis.

## CAUTION

The Miami LSO brace is a supportive brace only, and is not intended or guaranteed to prevent spinal injury. It should be applied by a licensed healthcare practitioner and as directed by a physician or other qualified medical authority. Össur accepts no responsibility for any injury which might occur while using the Miami LSO brace.

## SIZE SELECTION

Select the appropriately sized Anterior and Posterior Panels based on waist circumference, and the desired degree of motion restriction.

## SIZING CHART

### *Miami LSO Belts*

Size	Waist Circumference, inches (centimeters)	
Universal Belt (6" or 8" high)	26–50 (66–127)	
Optional Belt Extension	50–70 (127–178)	

### *Posterior Panels without Lateral Support*

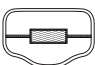
Size	Waist Circumference, inches (centimeters)	
Small	26–34 (66–86)	
Medium	34–42 (86–107)	
Large	42–50 (107–127)	
XLarge	50–70 (127–177)	

## Posterior Panel with Lateral Support

Size	Waist Circumference, inches (centimeters)	
Small	26–34 (66–86)	
Medium	34–42 (86–107)	
Large	42–50 (107–127)	
XLarge	50–70 (127–177)	

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Size	Panel Height, inches (centimeters)	
Slim	6 (15)	
Standard	8 (20)	

## BRACE APPLICATION

**Note:** Össur's recommended application technique is with the patient in a supine position.

1. Log roll the patient on to their side.
2. Position the Posterior Panel centered over the spine, with the bottom of the Posterior Panel at approximately the sacrococcygeal joint.
3. Log roll the patient back into the supine position.
4. Wrap the lumbar belt around the waist and through the Össur Fit Tool (**Fig. 2**). Adjust the length of the belt arms to position the tool centrally. Secure the hook and loop Stay Fit Tabs to desired sizing (**Fig. 3**).
5. Remove Össur Fit Tool and Adjustable Belt from patient (**Fig. 4**).
6. Place belt ends into the Overlap Closures (**Fig. 5**). To obtain the lowest profile fit, the belt arm ends can be trimmed if later adjustments are not required.
7. Once belt ends are secured in alligator tabs, the Overlap Closures will serve as belt ends (**Fig. 6**).  
**Note:** Panels can be customized by heat-molding, grinding, or trimming, to achieve correct patient fit (**Fig. 7**).
8. Adjust the Compression System Cord length if necessary (**Fig. 8**).
9. The Posterior Panel should be centered on the spine, with the bottom of the panel at approximately the sacrococcygeal joint (**Fig. 9**). The Anterior Panel should be centered on the abdomen with the bottom edge just above the symphysis pubis, while still allowing the patient to sit comfortably (**Fig. 10**).
10. Close belt securing the Overlap Closures. Slide thumbs through the holes in the two Compression System Handles, and pull until the brace is at the appropriate tightness. For best results, re-tighten the compression system while the patient is supine (**Fig. 11**).  
**Note:** If an Össur Fit Tool is not available, the sizing guide (**Fig. 12**) can be used to estimate best patient fit based on waist circumference. Simply fold or trim the belt arms at the suggested location. If the patient's waist exceeds 50" (127 cm), an extension belt can be added to each belt arm to reach a maximum belt circumference of 70" (178 cm).

## BRACE REMOVAL

1. Detach Compression System Handles slowly from the belt to loosen, and re-attach them at the original position.
2. Detach the Overlap Closures and remove the brace.
3. To ensure a proper fit, ensure the compression system is stretched to its full width before re-applying the brace.

## BRACE ADJUSTMENTS

- The Miami LSO brace can be easily customized to the needs of your patient.
  - Anterior and Posterior Panels can be added or removed throughout the rehabilitation period depending on desired motion restriction and compression
  - The Anterior and Posterior Panels can be modified as necessary to optimize patient fit and comfort. Simply remove the panels from their fabric sleeve and adjust the rigid panel with a heat gun (350°F/175°C). The panels can also be ground, and/or trimmed for proper relief and comfort.
  - Position the posterior and/or anterior panels in an optimal position on the belt for appropriate sagittal and coronal plane support.
- The length of the belt and compression cord can be adjusted to achieve a custom circumferential fit.

## PATIENT INSTRUCTIONS

- The patient should be instructed how to tighten and loosen the compression system while at home. Simply have them slip their thumbs in the compression system handles, and detach them from the belt. At this point the handles can be pulled for a tighter fit, or relaxed for a looser fit.

**Caution:** Do not over-tighten the compression system to the point where it causes discomfort or difficulty breathing.

- For sitting, the patient may find it desirable to slightly loosen the compression system.

**Warning:** If the brace causes discomfort or skin irritation, discontinue the use of the product and contact your healthcare provider for further instructions.

## BRACE MAINTENANCE

The Miami LSO brace is designed for minimum maintenance and care.

- Remove the panel covers
- Hand-wash belt and/or cover(s) using mild soap and cold water
- Air dry flat
- Do not machine wash or place in tumble dryer

## PRODUCT MATERIAL LIST

Nylon, Polyamide, Spandex, Thermoplastic Polyurethane, Polyurethane, Ester Polyurethane, Polyester, Polyurethane Foam, Bamboo, Acrylonitrile Butadiene Styrene.

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