







Instructions for Use



---

UNLOADER ONE<sup>®</sup> X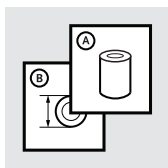

		3
EN	Instructions for Use	5
DE	Gebrauchsanweisung	8
FR	Notice d'utilisation	11
ES	Instrucciones para el uso	14
IT	Istruzioni per l'uso	17
NO	Bruksanvisning	20
DA	Brugsanvisning	23
SV	Bruksanvisning	26
FI	Käyttöohjeet	29
NL	Gebruiksaanwijzing	32
PT	Instruções de Utilização	35
PL	Instrukcja użytkowania	38
CS	Návod k použití	41
JA	取扱説明書	44
ZH	中文说明书	47
KO	사용 설명서	50
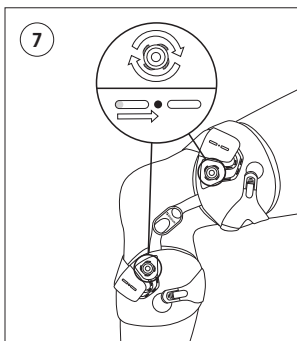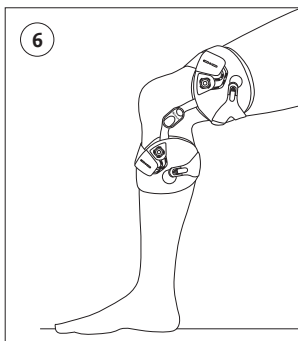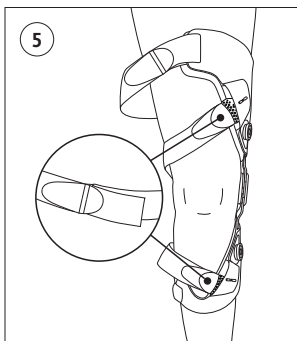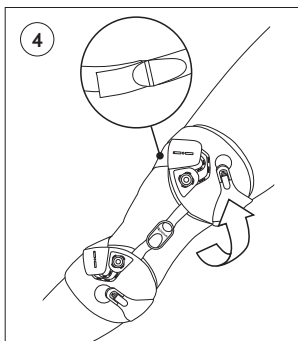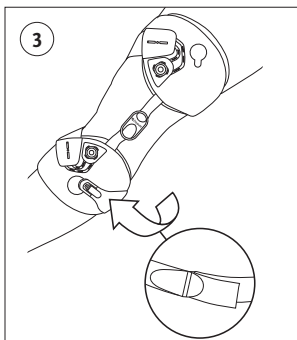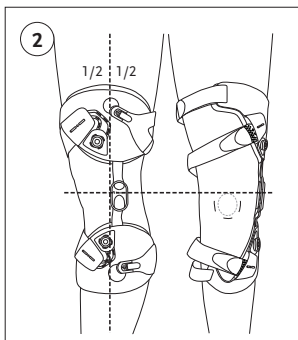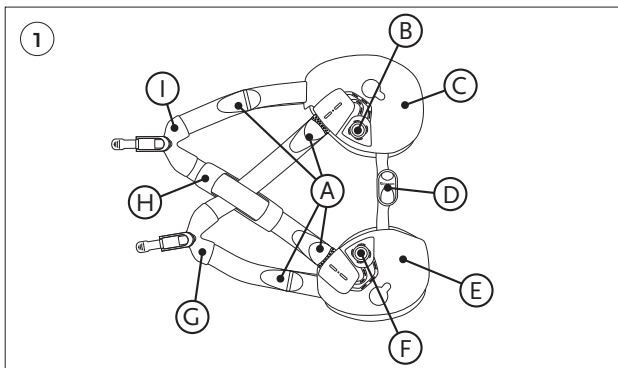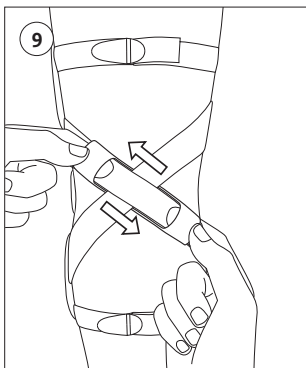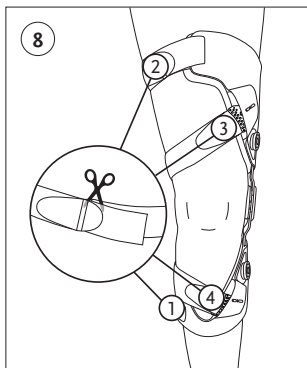
## UNLOADER ONE X

### LIST OF PARTS (Figure 1):

- A. Alligator clips
- B. Upper SmartDosing system
- C. Thigh shell
- D. Polycentric hinge
- E. Calf shell
- F. Lower SmartDosing system
- G. Calf strap and buckle with upper Dynamic Force System (DFS) strap
- H. Popliteal pad
- I. Thigh strap and buckle with lower Dynamic Force System (DFS) strap

### INDICATIONS FOR USE

The Unloader One X knee brace provides pain relief from osteoarthritis and degenerative meniscal tears through unicompartamental unloading of the knee.

Unloader One X is for people diagnosed with:

- Moderate to severe unicompartamental osteoarthritis of the knee
- Degenerative meniscal tears
- Other knee conditions that may benefit from unicompartamental unloading

### SAFETY INSTRUCTIONS

Regular practitioner supervision is recommended for users with peripheral vascular disease, neuropathy, and sensitive skin.

Care should be taken not to overtighten the brace.

For first time users, it is recommended to gradually increase usage time while building up tolerance to the brace.

Make sure the brace fits properly to minimize the possibility of skin irritation.

Unloader One X is a field adaptable knee brace. Adjustments to fit individual patient requirements should only be performed by an orthotist or qualified practitioner.

The brace can only achieve effective pain relief if fitted properly.

This device has been designed and tested based on single patient usage. Do not use for multiple patients.

If any problems occur with the use of this product, a medical professional should be contacted.

### BRACE APPLICATION

- **Figure 2:** Open upper and lower Quick Fit buckle.  
*Height positioning:* Align Össur logo on the hinge with upper edge of knee cap.  
*Side positioning:* Front edge of hinge cap should be at midline of leg.

- **Figure 3:** Ask user to sit down and extend leg while positioning the brace. The Quick Fit buckles and corresponding keyholes are color matched. Place lower buckle button into calf shell keyhole (blue) and use the palm of the hand to snap the buckle closed. Position calf strap above widest part of gastroc, fold strap at appropriate length into alligator clip and fasten strap so that it keeps brace securely and correctly positioned on leg.
- **Figure 4:** Place upper buckle button into thigh shell keyhole (yellow) and use the palm of the hand to snap the buckle closed. Adjust thigh strap to appropriate length by tensioning around leg and folding into alligator clip.
- **Figure 5:** With both SmartDosing dials at lowest setting, adjust the upper DFS strap length until it sits firmly against the leg, and then fold it into alligator clip. At this point, user should not experience any tension or unloading. Adjust the lower DFS strap in the same way.
- **Figure 6:** Ask user to bend knee with the foot flat on the floor.
- **Figure 7:** Turn upper and lower SmartDosing dials clockwise until indicators are in the middle slot.
  - Have user stand up and take a few steps to verify correct positioning of the brace and tightness of the straps.
  - Determine optimal DFS strap tension based on user's pain relief feedback.
  - If user requires more or less tension with indicator at mid setting, adjust the length of the DFS straps accordingly.
  - Aim for final SmartDosing setting at middle indicator slot since this will give user ability to adjust dosing during daily life activities.
- **Figure 8:** When final fit has been confirmed, trim the straps to the appropriate length starting with the calf strap so that brace sits correctly on leg while trimming other straps.
- **Figure 9:** With user in standing position, insert fingers into popliteal pad openings and move to an ideal position in popliteal area.

## BRACE REMOVAL

1. Ask user to sit with knee flexed and foot flat on floor.
2. Open both upper and lower buckles.
3. Pull buckle buttons out of keyholes.

## CLEANING

Insert entire brace into designated washbag, without removing pads or straps.

Machine wash using a gentle wash cycle.

Wash with a gentle detergent.

**NOTES:**

- Avoid contact with chlorinated water or salt water. In case of contact, rinse well with tap water.
- Do not wash using bleach as this will affect durability and may cause skin irritation.
- Do not wash using fabric softener.
- Do not tumble dry.

**PRODUCT MATERIAL LIST**

Aluminum, silicone, nylon, stainless steel, acetal, polypropylene, thermoplastic elastomer, polycarbonate, polyester, polyurethane, spandex, lithium grease, acrylic adhesive.

**DISPOSAL**

All components of the product and packaging should be disposed of in accordance with respective national environmental regulations. Users should contact their local governmental office for information on how these items can be disposed of in an environmentally sound manner.

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