medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 Unit 4/13-21 Hallmark Street
 Pendle Hill NSW 2145
 Australia
 T +61 2 9890 8696
 F +61 2 9890 8439
 sales@medi.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium NV
 Staatsbaan 77/0099
 3945 Ham
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Canada Inc / médi Canada Inc
 104-1375 Lionel-Boulet,
 Varennes, Québec,
 QC Canada J3X 1P7
 T +1 450-583-3317 / +1 800 361 3153
 F +1 888-583-6827
 service@medicanada.ca
 www.medicana.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

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medi Trading (Shanghai) Co. Ltd.
 Room 613, HnR Group Building
 No.1178, Tian Yao Qiao Road
 Xuhui District, Shanghai, 200232
 People's Republic of China
 T +86-21 50582318
 infocn@medi.cn
 www.medi.cn

Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS
 Mustafa Kemal Mah. 2156 Sokak No: 16 /4-5
 06510 Çankaya Ankara
 Turkey
 T: +90 312 435 20 26
 F: +90 312 434 17 67
 info@medi-turk.com
 www.medi-turk.com

UK Responsible Person
 medi UK Ltd.
 Plough Lane
 Hereford HR4 0EL
 Great Britain
 T +44-1432 37 35 00
 F +44-1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

protect.Maternity belt

Schwangerschaftsstützgurt · Support belt to reduce back pain during pregnancy

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso. Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning Användning. Návod k použití. Uputa za uporabu. Инструкция по использованию. Kullanma kılavuzu. Instrukcja zakładania. Használati útmutató. دليل الاستخدام. Uputstvo za upotrebu. Instrukciuni de utilizare. הוראות שימוש. Uzvilksanas instrukcija.

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Important notes

This medical device **[MD]** is made only for multiple uses on a single patient. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If you experience excessive pain or any discomfort whilst wearing, remove the product and immediately contact your doctor or medical supply store. Only wear the product on unbroken skin or over injured skin that has been medically treated, not directly on injured or damaged skin, and only after previous consultation with a medical professional.

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protect. Maternity belt

Intended purpose

protect. Maternity belt is a brace for stabilising  the lumbar spine using pressure that can be applied at different levels.

Indications

All indications which require stabilisation of the lumbar vertebrae with a range of pressure levels, such as:

- Pain in the lumbar region during pregnancy
- Lower back pain

Contraindications

Inguinal hernia

Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Intended users and patient target groups

Healthcare professionals and patients, including persons who play a supporting role in care, are the intended users for this device, provided that they have received briefing on this from healthcare professionals.

Patient target group: Healthcare professionals should provide care to adults and children, applying the available information on the measurements/sizes and required functions/indications, in line with the information provided by the manufacturer, and acting under their own responsibility.

Instructions for putting on

Please wash before first use.

- Place the maternity belt around your hips and position it so that the stabilisation roads are located over your spine in the middle of your lower back. Next fasten the two ends at the front under your abdomen (Fig. 1). Make sure that the belt is positioned well and does not bother you when walking or sitting.
- To fasten the elastic tension straps, place your hands in the loops of the two tension straps on the outermost side (Fig. 2). Pull these forwards until you reach the desired level of tension. Using the Velcro fastening, gently press the straps to attach them to the belt.

Care instructions

Hook and loop fasteners should be closed for washing. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material and cause skin irritations.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.

- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

Material composition

Polyester, cotton, polyamide, elasthan, synthetic leather.

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

Dispose of properly after use.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR).

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