


# Steel Rolling Walker


## 5 Adjusting the Handle Height

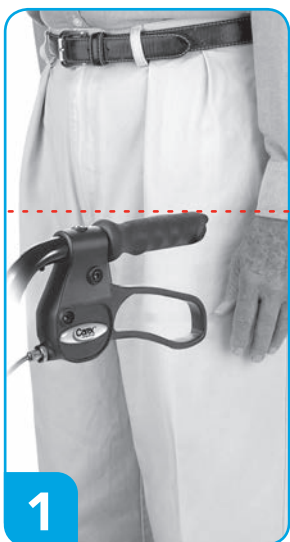

## Using the brakes



Pull up to apply brakes

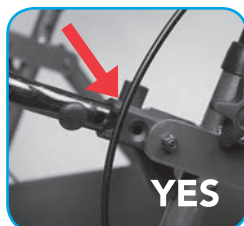


Release brakes

**Note:** Cable needs to be placed outside seating area and frames.



Push down to lock brakes

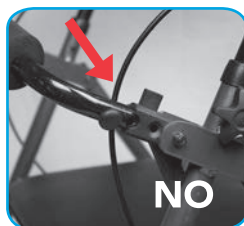

## 1 Check Parts

Backrest




Rolling walker frame



Handles with brake assembly and back wheels x2



Front wheels

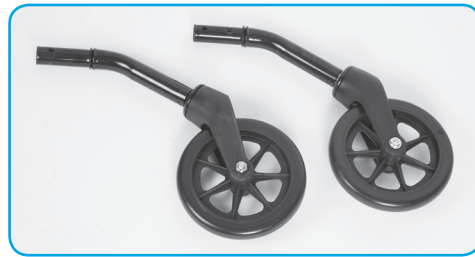

E-clips & hand screws with washers




Note: actual items may vary slightly from what is shown.

## 2 Handle Assembly



1

Cable should be outside frame.

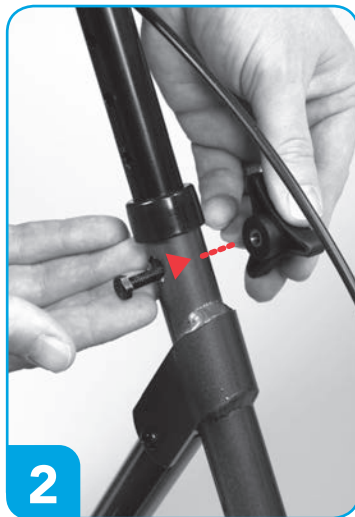

2

**NOTE:** Hex shaped bolt should fit onto hex shaped hole in tubing. Hand screw faces out.

## 3 Front & Rear Wheel Assembly



1

Insert back and front wheels using E-Clip and hand screw to secure them into place. Make sure all knobs are firmly tightened before use.



2



3

## 4 Backrest Assembly & Braking System



1

Push in side buttons and insert backrest. Flip up backrest for compact storage.



2