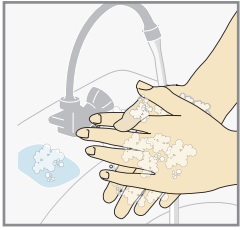

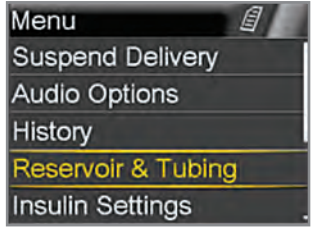
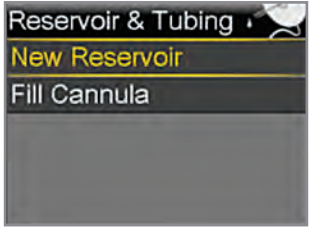




CHANGING THE SILHOUETTE® INFUSION SET: MINIMED® 630G INSULIN PUMP



START HERE:



1  Wash your hands. Press .

2  Select **Reservoir & Tubing**.

3  Select **New Reservoir**.

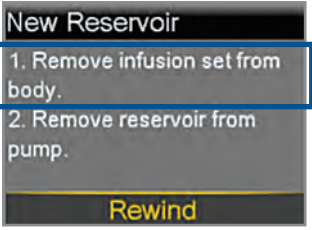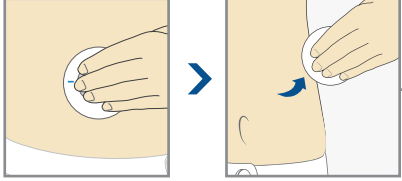
4  Remove the infusion set you have been using by loosening the adhesive and pulling away from body. 

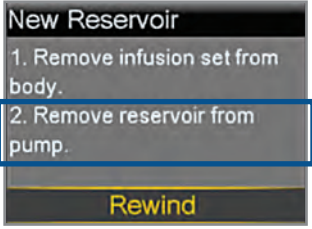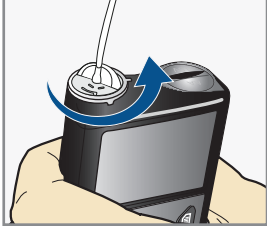
5  Remove the used reservoir from the pump. 

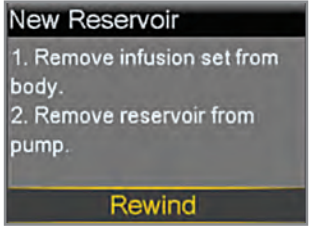
6  Select **Rewind**. 


FILL RESERVOIR & CONNECT TO THE INFUSION SET TUBING

Follow the next steps to fill reservoir with insulin and connect to the infusion set tubing.

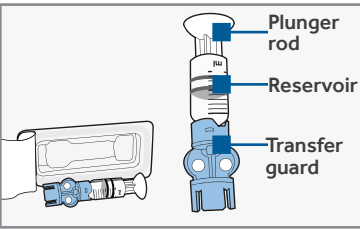1  Remove from package. Make sure insulin vial is at room temperature to reduce the risk of air bubbles.

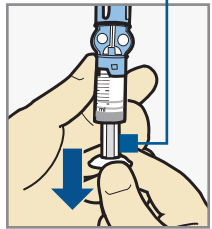2  Pull plunger down to the amount that you plan to fill with insulin.

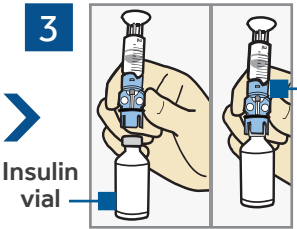3  Wipe vial with alcohol. Place vial on table. Firmly press the blue transfer guard onto vial.

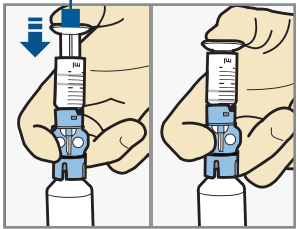4  Push and hold plunger down.

Continued on next page



CHANGING THE SILHOUETTE® INFUSION SET: MINIMED® 630G INSULIN PUMP

Warning: Do not use the reservoir or infusion set if any liquid gets on the top of the reservoir or inside the tubing connector. Liquid can temporarily block the vents and may result in the delivery of too little or too much insulin, which can cause hypoglycemia or hyperglycemia.

5

Insulin vial
Plunger

With your thumb still on the plunger, flip over so vial is on top. Release thumb and pull plunger down to fill with insulin.

6

Tap the reservoir to move air bubbles to top of reservoir. Push plunger up to move air into vial.

7

If needed, pull plunger back down to amount of insulin needed for 2-3 days.

8

Reservoir
Transfer guard

To avoid getting insulin on the top of the reservoir, turn vial over so it is upright. Hold transfer guard and turn reservoir counter-clockwise and remove from transfer guard.

CONNECT RESERVOIR TO INFUSION SET

You will place the tubing connector onto the end of the infusion set to the filled reservoir.

1

Site connector
Tubing connector
Insertion piece

Remove infusion set from package.

2

Connector

Gently push the tubing connector onto reservoir. Turn clockwise until locked. You will hear a click.

3

If you see air bubbles, tap reservoir to move them to top. Push plunger just a bit to move them into tubing.

4

Plunger

Twist plunger counter-clockwise to loosen and remove.

THE BACKLIGHT MAY HAVE TURNED OFF

Press any button to turn the screen back on.

Active Insulin 0.0 u
Load Reservoir

Select **Load Reservoir** and unlock pump if necessary.

New Reservoir

1. Fill reservoir.
2. Connect tubing to reservoir.
DO NOT CONNECT TO BODY.

Next

Select **Next**.

Continued on next page



CHANGING THE SILHOUETTE® INFUSION SET: MINIMED® 630G INSULIN PUMP

New Reservoir


3. Place reservoir into pump and lock.

DO NOT CONNECT TO BODY.

Next

1

➤

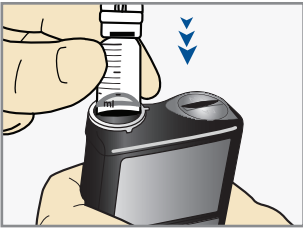

Place reservoir into pump.

2

➤



Turn clockwise until you feel reservoir lock into place.

3

➤



Press the edges of the site connector. With the other hand, pull the cap off.

4

➤

New Reservoir

3. Place reservoir into pump and lock.

DO NOT CONNECT TO BODY.

Next

Select **Next**.

LOAD RESERVOIR AND FILL TUBING

Follow these steps to load the reservoir and fill the tubing.


1

➤

Load Reservoir

Select Load and hold until complete. DO NOT CONNECT TO BODY.

Load Next

Select **Load** and keep holding .

2

➤

Load Reservoir

Complete

DO NOT CONNECT TO BODY.

Load Next

When you see this screen, select **Next**.


3

➤

Fill Tubing

DO NOT CONNECT TO BODY. Hold Fill until drops appear. Then select Next.

Fill Next

Select **Fill** and keep holding  until you see drops at the end of tubing, then let go.


4


➤

Fill Tubing

DO NOT CONNECT TO BODY. Hold Fill until drops appear. Then select Next.

Fill Next

After you see drops, press  and select **Next**.

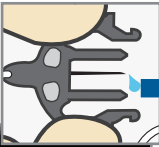

Drops at end of tubing

INSERT INFUSION SET (Two Methods)

Next, follow the steps to insert the infusion set into your body.


Fill Cannula?

1. Insert infusion set into body.

2. Select Fill to fill cannula or Done if not needed.

Fill Done

➤

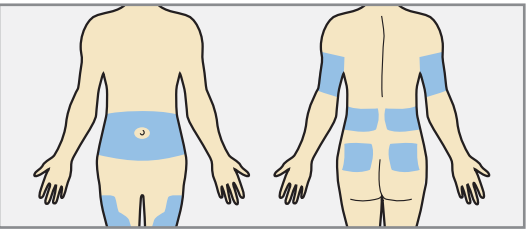

Choose an insertion site from the shaded areas shown here. Wipe with alcohol or other antiseptic.


PROCEED TO METHOD 1 or METHOD 2

CHANGING THE SILHOUETTE® INFUSION SET: MINIMED® 630G INSULIN PUMP

METHOD 1:


By hand

1

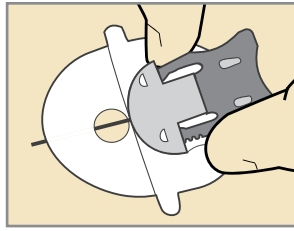


Hold the insertion piece by the gray needle handle using thumb and middle finger.

2

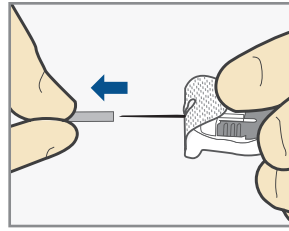



Hold the tape with your forefinger and remove the needle guard.

3

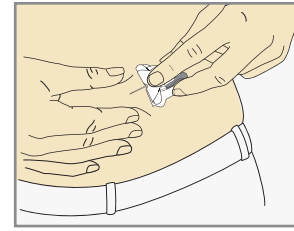

Pinch up skin at prepared site. Use other hand to insert needle at a 20-45 degree angle.

4

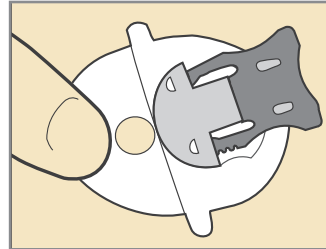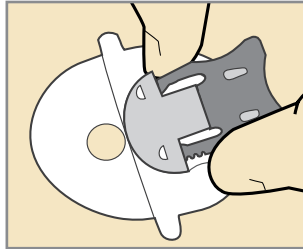


After it is inserted release the skin and hold inserted piece in place. **Now go to the REMOVE INTRODUCER NEEDLE section on page 5.**

METHOD 2:


Using the serter

1

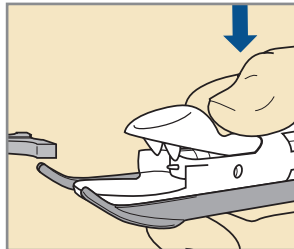

Use thumb to press down on the Lock & Release handle of the serter.

2

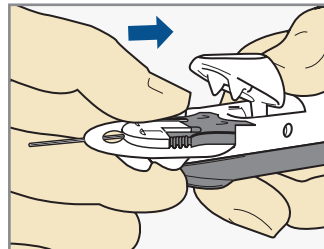


Place the insertion piece inside serter and let go of the Lock & Release handle. The serter prongs should fit securely into the holes of the insertion piece.


Continued on next page




CHANGING THE SILHOUETTE® INFUSION SET: MINIMED® 630G INSULIN PUMP

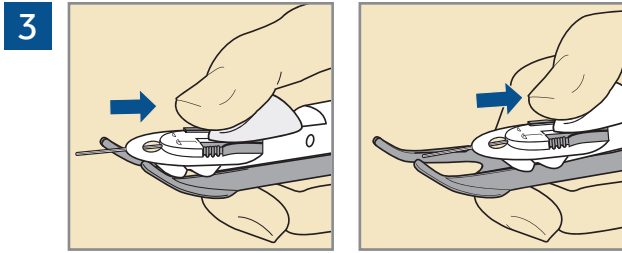


Use thumb to slide the Lock & Release handle back until you hear a click.

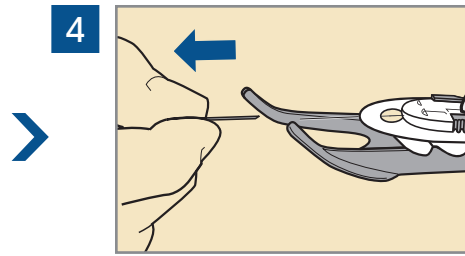


Gently pull off needle guard.

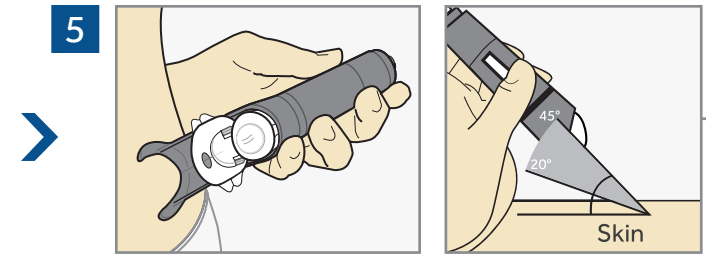

Rest arms ofserter against skin so that it is at a 20-45 degree angle.

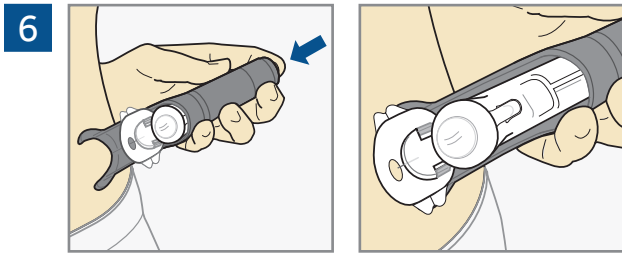


Press the white button at the top ofserter to insert the infusion set.

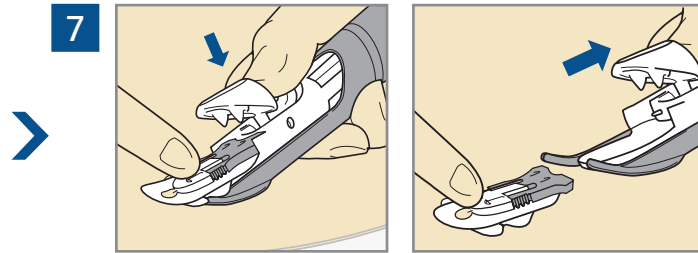


Hold infusion set against skin with one hand. With the other hand press the Lock & Release handle to openserter. Then, slideserter back from infusion set.

REMOVE INTRODUCER NEEDLE
Next, follow the steps to remove the introducer needle.

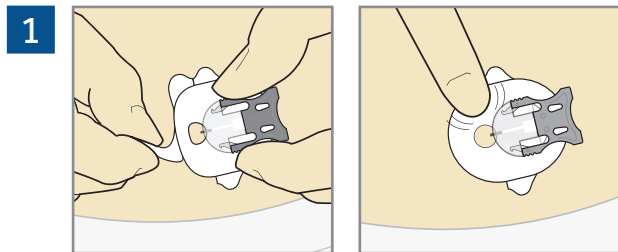

Peel paper from the front part of the adhesive and press adhesive securely to skin.

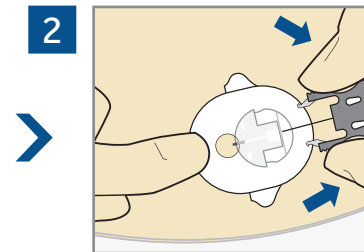


Squeeze arms on the gray needle handle and pull back to remove introducer needle.

DISPOSE NEEDLE INTO A SHARPS CONTAINER


Continued on next page



CHANGING THE SILHOUETTE® INFUSION SET: MINIMED® 630G INSULIN PUMP

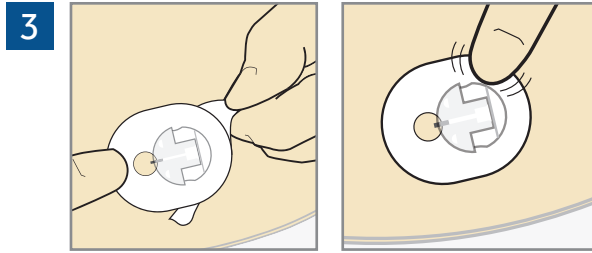


3 Peel rest of the paper without lifting infusion set too far from skin. Press adhesive securely to skin.

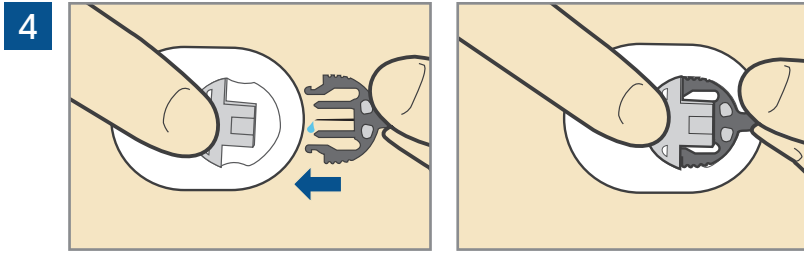

4 Connect infusion set tubing by sliding the site connector into the insertion piece on your skin.

FILL CANNULA
You will now fill the cannula, the little tube under your skin, with insulin.

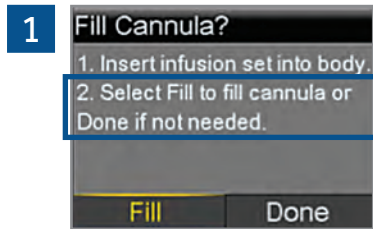

1 Select **Fill**.



2 Select **Fill amount** and enter 0.700. Then press **○**.




Note: Your pump will remember the **Fill amount** that you used last. Always verify that the **Fill amount** is correct.

- If it is correct, press **∨** to **Fill Now** and press **○**.
- If it is incorrect, press **○**. Change to correct amount and press **○**. Select **Fill Now**.



3 Select **Fill Now**.

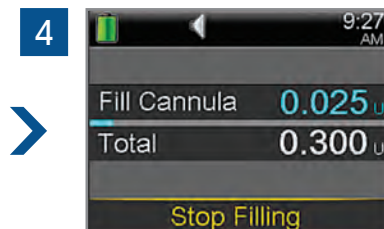

4 The Home screen displays the insulin as it fills the cannula.



Note: Select **Stop Filling** if you need to stop, for example, if you notice the **Total** amount is incorrect. This should rarely happen if you have verified the **Fill amount** on the previous screen.

YOUR INFUSION SET CHANGE IS NOW COMPLETE!