



The Nisus® Negative Pressure Wound Therapy

PURPOSE

Negative Pressure Wound Therapy (NPWT) has become a preferred choice of care for treating acute and chronic wounds. The The Nisus® may be used both at the hospital and at home, easing transition and facilitating continuity of care. The ease of use, quiet operation and compact size makes it a device that can be carried in a discreet bag, facilitating mobility. The docking station allows for easy recharging of the unit. Therapy data can be checked on the built-in LCD screen or stored and uploaded to a PC.

INDICATIONS FOR USE

- Diabetic/Neuropathic ulcers
- Pressure or venous insufficiency ulcers
- Chronic wounds
- Traumatic wounds
- Acute or sub-acute wounds
- Dehisced wounds
- Partial thickness burns
- Flaps and grafts

CONTRAINDICATIONS

- Malignancy in the wound
 - Untreated osteomyelitis
 - Non-enteric & unexplored fistulas
 - Necrotic tissue with eschar present exposed vasculature
 - Exposed vasculature
 - Exposed nerves
 - Exposed anastomotic site
 - Exposed organs
 - Malnourished patients
 - Pediatric use
-

CAUTIONS

- Federal law restricts this device to sale/rental by or on the order of a physician.
- These instructions for use are a general guide. Medical matters must be addressed by a qualified medical professional.
- Nisus® products run through a quality management system. In order to ensure their safe and proper operation, please meet the conditions below. (Failing to do so will void the warranty).
 - o Service work may only be carried out by Nisus® authorized personnel.
 - o Before use, please consult the indications for use and warnings, the precautions as well as the contraindications. This will provide you with important information about using the unit. Non-observance can lead to considerable dangers.
 - o Before you plug in the device, please check that your local power supply is the same as the voltage given on the specification plate.
 - o The Nisus® Negative Pressure Wound Therapy System is approved exclusively for the use described in these instructions for use.
 - o The Nisus® Negative Pressure Wound Therapy System is not for use in pediatric patients.
- **Please note:** For use only by medically trained persons who have been adequately trained in suction procedures, wound care, negative pressure wound therapy and in the use of aspirators. Therapy or setting changes (vacuum, mode...) may only be done under order of a physician or your qualified caregiver.
- **INCORRECT USE MAY CAUSE PAIN AND INJURY**